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

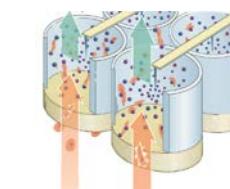
Module 1
PlasmerisationTM
Microbial
Destruction

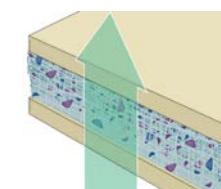Destruction of microorganisms by exposure to strong electric fields and oxidizing species created in unique non-thermal plasma chambers


Module 2
Biological
econtamination
and HEPA
particulate
reduction

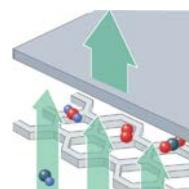Highly charged matter from the plasma chambers is captured by an electrified collection media. Organic material is continually exposed to the plasma ions to ensure total biological decontamination.

Module 3
Molecular
conversion

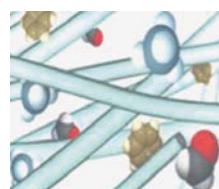Removal of
oxidative chemicals
by a catalytic
converter.

Module 4
Molecular
adsorption

Organic volatile
molecular
pollutants are
adsorbed onto an
activated carbon
medium.

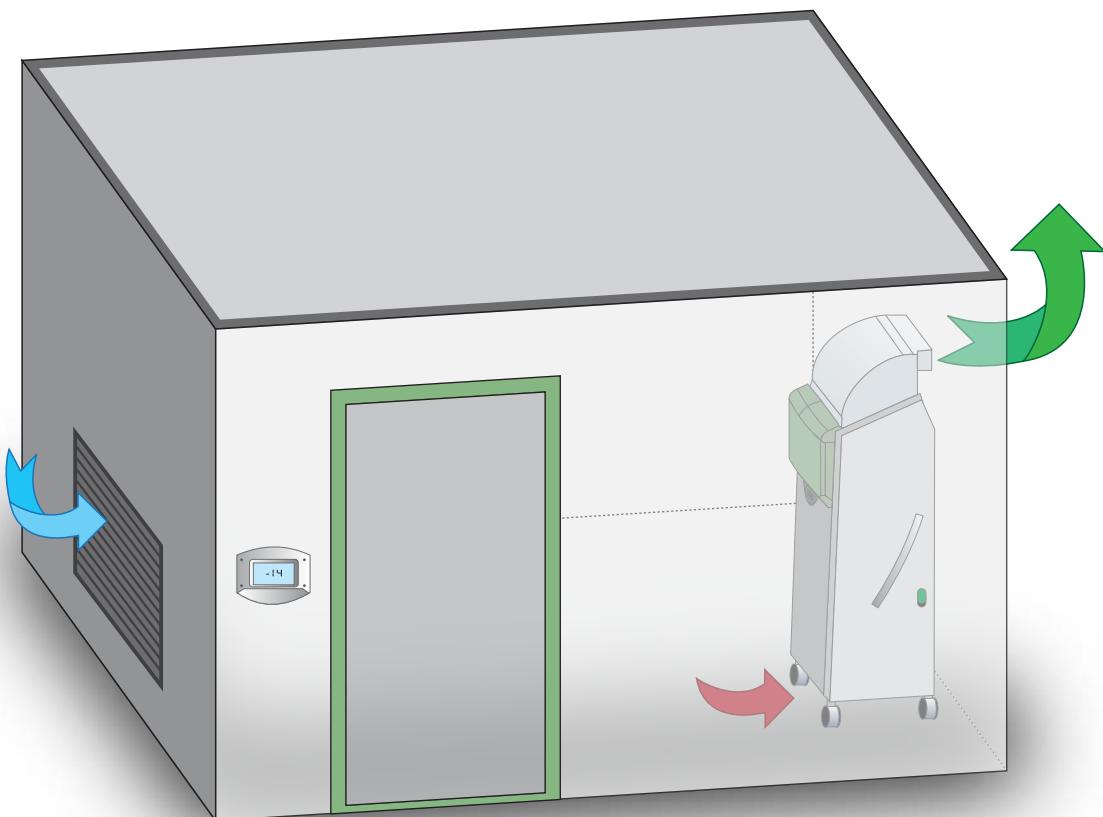
- **Eliminates risk associated with the storage and disposal of viable microorganisms.**
- **Decreases the concentration of volatile organic compounds (VOCs) in air.**
- **Maintains high airflow through media with less noise, and lower power consumption.**

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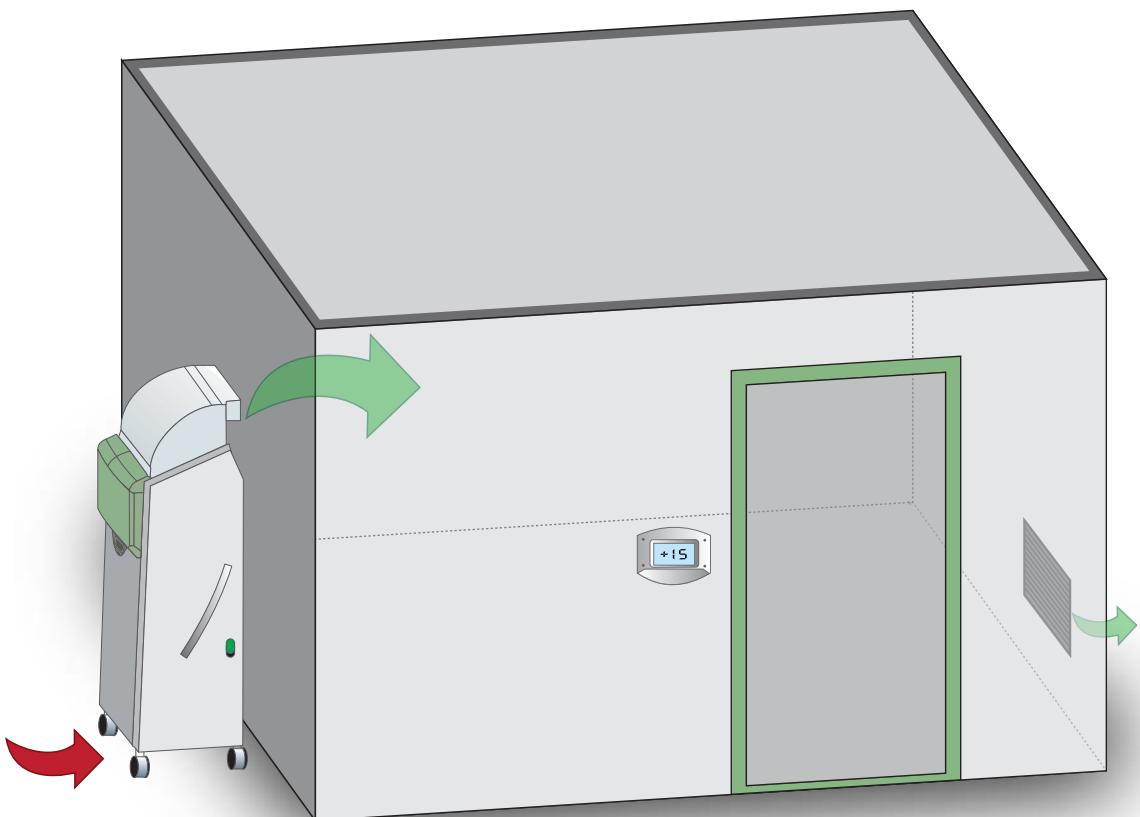
Adjustment and control through large touch screen.

Modular isolation room for Airborne Infectious Isolation Room (AIIR) or Protective equipment application.

Performance

- 15 to 30 Pa of pressure differential
- ISO 5 (press+)/ ISO 8(press-)
- Less than 45 dB (A)

Modular

- Cleanroom-grade wall panels

Ergonomic

- Automated door with controlled air flow leakage
- Integrated lighting
- Touch screen for complete monitoring and control safety
- Hepa-MD™ microbial destruction
- Datalog of operating parameters
- Air treatment in the close vicinity of Biocair serves as regular anteroom.

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Immunair iso:mobile or stationary protective environment for fast set-up of very high risk controlled areas (ISO 5)

Deployable plenum to protect immunocompromised patients in BMT transplant, oncology and immunology units.

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MDP and FDP dimensions

Closed (MDP)	H 127×W 80×L210
Deployed (MDP/FDP)	H 214×W210×L271
Protected zone (MDP/FDP)	H 192×W180×L235
Indicative weight (MDP/FDP)	350 kg / 120kg

Technical specifications

	Pa T2006	Sentinel
Airflow rate		
Night mode	500 m ³ /h	400 m ³ /h
Intermediate mode		600 m ³ /h
Day mode	1100 m ³ /h	850 m ³ /h
Sound Levels under the Plenum		
At 400 m ³ /h		39 dB(A)
At 500 m ³ /h	42 dB(A)	
At 600 m ³ /h		43 dB(A)
At 850 m ³ /h		48 dB(A)
At 1100 m ³ /h	51 dB(A)	
Air changes per hour/max		
	Up to 135 ACH	Up to 104 ACH

CASE STUDY

Multicenter Clinical Study:
4 Hospital Hematology/Oncology Wards 12 month study

- Microbiological performance achieved (> 1000 samples):
 - Less than 10CFU/m³ for Total Flora.
 - Less than 1 CFU/m³ for Fungal Flora.
 - Data statistically validated and sourced from actual clinical studies incorporating real patients.
- Particulate performance achieved:
 - ISO 7 outside the patient tent (starting with Initial contamination in a standard room ISO 9).
 - ISO 5 within the patient tent (once ISO 7 achieved in the external room area).
- Decontamination kinetics:
 - 90% reduction of contamination within 4 minutes at 500 m³/h .
 - 90% reduction of contamination within 2 minutes at 1100 m³/h.
 - Data from hematology department studies.

PLASMAIR™ Sentinel

Light and easy solution for securing high infectious risk areas

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Friendly user interface
with led technology.

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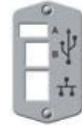
Easy access to the menus by touching the relevant graphics.
5 available languages.

Connection availability to Central Management System.

Performance Range

Bacteriological Classification	Total flora : < 10 CFU/m³ based on an ARH of 18 vol/h Fungai : < 1 CFU/m³ based on an ARH of 12 vol/h
Airborne Particulate Cleanliness Class	Iso 7 based on an ARH of 12 vol/h
Time to achieve 90% particulate decontamination Kinetic in minutes for particles of diameter ≥ 0.5 µm	-<12min at an ARH of 15 vol/h -<7min at an ARH of 30 vol/h

Dimensions and weight

Height	194cm
Width	91cm
Depth	69cm
weight	191kg

Technical Specifications

Airflow rates	500-1000-1500-2000-2500 m³/h		
Max electrical power	600VA/600 W		
Sound levels (according to standard NF-EN ISO 3744:2012 at 1m/2m)	-500 m³/h	30dB(A) / < 30dB(A)	
	-1000 m³/h	39dB(A) / 35 dB (A)	
	-1500 m³/h	47dB (A)/43dB(A)	
	-2000 m³/h	53dB(A)/49dB(A)	
	-2500 m³/h	58dB(A)/54dB(A)	
Frequency and voltage	~ 100V , ~ 110/120V , 50/60/Hz		

HEPA Guardian

Friendly user interface
with led technology

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Performance range

Airborne particulate cleanliness class	ISO 7 based on an ARH of 12 vol/h
Bacteriological classification	M10 based on an ARH of 12 vol/h
Aerosol filtration efficiency at 1000 m ³ /h	>99.997% Particless ≥ 0.3 μm

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high filtration efficiency systems

Performance range

Airborne Particulate Cleanliness Class	ISO 7 based on an ARH of 14
Microbiological classification	M10 based on an ARH of 14 vol/h
Aerosol filtration efficiency at 600 m ³ /h	>99.99% Particles ≥ 0.3 μm

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Removal of airborne volatile organic
Components (VOC) in invitro fertilization lab

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Technical specifications

Air volume treated	600-800-1,200-1,800/h (353 – 1,060 cfm)
Electrical consumption	240 VA
Electrical supply	V/230V -50Hz/60Hz 100

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- No additional adjuvants (Ag, formaldehydes essential oils...)
- No solid residue after treatment

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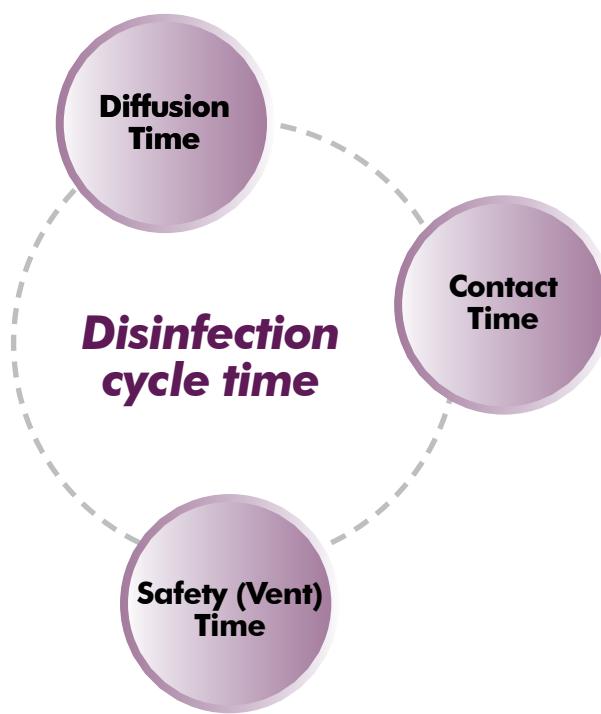
**Complies with the standard NF-T72-281 (november 2014)
achieved in accredited and independant laboratories (COFRAC)**

Specification

Surface treatment capacity (room volume)	830 m ³	
Volumetric flow rates of X-CID disinfectant product	18 ml/min	
Water/solid protection index	Complete unit	IP40
	Control panel	Sealed touch screen
Overall dimensions	H397xL561xW340 mm	
Weight	8 kg	
Max electrical power	1205 VA	
Electrical power supply	~200/240V ; 50/60 Hz	

Bacterical efficacy according to NF-T72-281:

Logaritmic reduction of

Standard protocol 3 ml/m ³ with a contact time of 1 hours Enhanced protocol 6 ml/m ³ with a contact time of 15 minutes - Escherichia coli - Staphylococcus aureus - Enterococcus hirae	>5 log
Fungicidal and yeasticidal efficacy CCORDING TO NF-T72-281: Standard protocol 3 ml/m ³ with a contact time of 1 hours Enhanced protocol 6 ml/m ³ with a contact time of 15 minutes - Cadida albicans - Aspergillus niger	>4 log
Virucide efficacy according to NF-T72-281: Standard protocol 3 ml/m ³ with a contact time of 1 hours - Enhanced protocol 6 ml/m ³ with a contact time of 15 minutes - Aspergillus niger	>4 log
Sporicidal efficacy according to NF-T72-281: Standar protocol 3 ml/m ³ with a contact time of 1 hours Enhanced protocol 6 ml/m ³ with a contacttime of 15 minutes - Bacillus subtilis - Clostridium difficile	>3 log
Tuberculocidaland mycobactericidal efficacy according to NF-T72-281: Standar protocol 3 ml/m ³ with a contact time of 1 hours Enhanced protocol 6 ml/m ³ with a contact time of 15 minutes - Mycobacterium terrae - Mycobacterium avium	>4 log

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